

TS 112 – TRICHOMONAS SELECTIVE SUPPLEMENT I

INTENDED USE

For selective isolation of *Trichoderma* species from clinical specimens.

COMPOSITION

Ingredients	Concentration
Streptomycin	500mg
Penicillin G	125000Unit

(per vial sufficient for 500 ml medium)

Applied in Medium


TM 1212 - KUPFERBERG TRICHOMONAS BROTH BASE (TRICHOMONAS BROTH BASE, KUPFERBERG)

INSTRUCTION FOR USE

Rehydrate the contents of 1 vial aseptically with 5 ml of sterile distilled water. Mix well and avoid frothing of solution. Aseptically add the contents to 475 ml of sterile, molten, cooled (45-50°C) TM 1212 - KUPFERBERG TRICHOMONAS BROTH BASE (TRICHOMONAS BROTH BASE, KUPFERBERG) along with 25 ml sterile beef or human serum. Mix well and dispense as desired.

STORAGE

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Vials should be stored in sealed container at 2-8°C.



NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

***For Lab Use Only**
Revision: 3 March., 2022