

TS 219 – CEFOXITIN SUPPLEMENT

INTENDED USE

For selective isolation of Methicillin Resistant *Staphylococcus aureus* from clinical specimens.

COMPOSITION

| Ingredients | Concentration |
|-------------|---------------|
| Cefoxitin | 3mg |

(Per vial sufficient for 500 ml medium)

Applied in Medium

TM 1635 - CHROMOGENIC MeReSA AGAR BASE


TM 1561 - MeReSA AGAR BASE

INSTRUCTION FOR USE

Rehydrate the contents of 1 vial with 5 ml of sterile distilled water and aseptically add to 500 ml of sterile molten TM 1635 - CHROMOGENIC MeReSA AGAR BASE / TM 1561 - MeReSA AGAR BASE. This supplement can either be used individually or in combination with MeReSa Selective Supplement for more selectivity. Mix well and pour into sterile petri plates.

STORAGE

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Vials should be stored in sealed container at 2-8°C.



NOTE: Please consult the Material Safety Data Sheet for information regarding hazards and safe handling Practices.

*For Lab Use Only
Revision: 8 March., 2022